
INSTRUCTIONS FOR USE

Date of issue: May 2013

- EasyTape® is suitable for use in pain management by relieving the concerning part of the body
- Every piece of tape can be used one time only; remove the tape slowly after use. After use, dispose the tape
- Clean the body parts that need taping; cut the opened tape when necessary; press the surface gently to the skin
- Apply the tape in the direction that feels most comfortable
- Taping of more complex body parts shall be performed by a medical specialist.
- Consult your doctor beforehand when having serious pain
- When using the product for a prolonged period of time a doctor shall be consulted
- The tape cannot be used over untreated wounds
- Be careful to use this tape in case of sensitive skin. In case of allergies remove the tape immediately
- Use for children only after consulting a medical doctor and under constant parental supervision (possible choking hazard: children nibbling on tape)
- For further information about the easytaping method: contact Msys BV or a reseller
- Dispose of the tape when it has lost its sticking effect
- Tape can be stored for 3 years after date of production when kept in a dry and cool place; expiry date: see packaging (year-month indication)
- Save the natural environment: recycle this paper and the carton box after use

Msys B.V., van Breugelplantsoen 42
3771 VR Barneveld, the Netherlands. www.msys.nl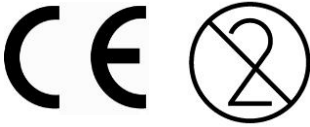

Distributed by Meducation, Marco Stekhoven
8008 Zürich, Switzerland. www.meducation.ch

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